

One Puff™ Aspirator / Resuscitator Kit

The **ONE PUFF™ ASPIRATOR / RESUSCITATOR KIT** is a simple, two-part, handheld instrument designed to clear breathing pathways and stimulate the first breaths of newborn puppies, kittens, and other small animals. **ONE PUFF™** is the safe, non-invasive, and effective alternative to the use of traditional methods, such as shaking, swinging, or mouth-to-mouth, which can cause irreparable harm to newborn animals. **ONE PUFF™** is an invaluable tool for all veterinarians, breeders, and pet owners and has saved hundreds of distressed newborn small animals, especially in cases of life-threatening airway obstruction following natural birth or C-section.

The **ONE PUFF™** is a controlled system that performs a specific function, depending on the mask used for the specific newborn small animal. The Aspiration Mask "A" is designed to administer a safe vacuum suction to remove mucus blocking the animal's airway. Respiration Mask "B" is designed to provide a small release of air to stimulate respiratory response. **ONE PUFF™** has been tested to consistently deliver 5 to 10 ml of air pressure, a level which has been proven safe for newborn small animals.

ONE PUFF™ ASPIRATOR / RESUSCITATOR KIT includes:

- Aspiration instrument and mask, unit labeled "A"
- Resuscitation instrument and mask, unit labeled "B"
- Carrying case with Instructions for Use
- Two-year McCulloch Medical product warranty



Item # 55325

Item	Product Description	UOM
55325	One Puff™ Aspirator / Resuscitator Kit, Puppy & Kitten	Each
55315	One Puff™ Aspirator / Resuscitator Kit, Raptor	Each

**EASY TO USE,
LIFE SAVING,
AWARD WINNING!**

One Puff™ Aspirator / Resuscitator Kit

HOW TO USE THE ONE PUFF™ ASPIRATOR / RESUSCITATOR KIT:

Prior to first use of the ONE PUFF™, clean in 1:20 bleach / water solution and dry thoroughly.

ASPIRATING



- To test the Aspiration Mask, use the end of the unit labeled "A". If the aspiration unit is functioning properly, there should be a slight vacuum sensation when holding the pre-assembled unit against user's arm while pumping the instrument.
- If the animal is not breathing after delivery or C-section, pick the animal up gently with head facing downward at a 45° to 90° angle from the horizon.
- Place mask labeled "A" securely over muzzle of animal, ensuring the muzzle is clearly in position to receive the air movement.
- Compress instrument trigger once to aspirate the animal, then remove mask.
- Gently massage the thoracic area, behind the front legs and in front of the stomach. Check for breathing reflex. If no breathing occurs, proceed to resuscitation.

RESUSCITATING



- To test the Resuscitation Mask, use the end of the unit labeled "B". If the resuscitation unit is functioning properly, there should be a slight release of air when holding the pre-assembled unit against user's arm while pumping the instrument.
- Place mask labeled "B" securely over muzzle of animal, ensuring the muzzle is clearly in position to receive the air movement.
- Compress instrument trigger once, then remove mask.
- Gently massage the thoracic area, behind the front legs and in front of the stomach. Allow a few moments for the animal to establish breathing reflex.
- If no breathing reflex is observed, repeat resuscitation process until successful breathing is achieved.

DO NOT pump the ONE PUFF™ instrument continuously while aspirating or resuscitating as this could have a detrimental effect on the newborn small animal. Only one or two puffs are necessary for stimulation of the animal's natural respiratory response.