

# Buy 1 Bottle of Sur-Vet® Secure Get 1 **FREE!**

Limit of 10 free bottles of Sur-Vet® Secure during promotion.



## Topical adhesive for catheter securement *and* site protection

- **Up to 3X stronger** securement than transparent film dressings alone and superior adhesiveness compared with mechanical securement devices<sup>2</sup>
  - Reduces catheter movement, migration, and dislodgement<sup>1</sup>
- **Effectively seals** the catheter insertion site
  - Can help reduce early dressing changes and eliminate 24-48 hour initial dressing change protocols<sup>3</sup>
  - Formulation reported to exhibit activity against gram-negative and gram-positive bacteria, yeast, and fungi eliminating greater than 8-logs after 3 minutes of contact in in-vitro studies<sup>4</sup>
  - Allows the direct visualization of insertion site at all times, unlike some antimicrobial dressings



Sur-Vet Secure adhesive can be used on peripheral IVs by placing 2-3 drops over the insertion site and underneath the catheter hub.

### FAQ's:

- **How many drops are required to secure a catheter? Where should the drops be placed?**
  - Apply one drop of adhesive directly on top of the catheter injection site. An additional 1-2 drops of adhesive can be used under the catheter hubs and/or wings for a stronger securement.
- **How is the Sur-Vet Secure adhesive removed?**
  - Standard adhesive remover or acetone is recommended. If removal is not required early, Sur-Vet Secure will naturally wear away, typically within 5-7 days, for easy removal of catheter.

Description	Sur-Vet Secure Catheter Securement Adhesive
Product Code	SV-300V
Fill Volume	2.5ml
Packaging	50 applications per bottle

## Don't miss this limited time offer!

To qualify, place your order for Terumo Sur-Vet® Secure **between December 1st through January 31st**

[Click here to view the Sur-Vet® Secure Training Module](#)

©2018 Adhezion Biomedical 08/18. All rights reserved. SPI-SS01-1801

1. Intravascular canine model 2017, reduced dislodgement, data on file.
2. Data on file. Average securement strength calculated from measured values. Nov/Dec 2016.
3. Kleidon, et al. A Pilot Randomized Controlled Trial of Novel Dressing and Securement Techniques in 101 Pediatric Patients. J Vasc Interv Radiol. 2017 Sep 18.
4. Prince, et al. Immobilization and Death of Bacteria by Flora Seal® Microbial Sealant. International Journal of Pharmaceutical Science Invention, 2017.

**RX ONLY.** The advertisement is directed to physicians only, and not to consumers. Refer to product labels and packaging insert for complete warnings, precautions, potential complications, and instructions for use. Products may not have regulatory approval in all countries. Please contact your local sales representative if you have questions about the availability of products in your area.

Terumo reserves the right to cancel promotion at any time.

TERUMO, TERUMO and SUR-VET are trademarks owned by Terumo Corporation, Tokyo, Japan, and are registered with the U.S. Patent and Trademark Office. ©2020 Terumo Medical Corporation 10/20. All rights reserved. PM-03899