

TRANSCEND™

Topical Wound Management System

TRANSCEND™

Topical Wound Management System

KINETIC™ TECHNOLOGIES 961 Beasley Street, Suite 270 Lexington, Kentucky 40509
877.786.9882 (Toll Free) 859.258.9160 (Phone) 859.258.9188 (Fax)

FEATURES

Hyaluronate Sterile Sodium solution
60 mg/6mL (10 mg/mL)

DESCRIPTION

Transcend™ is a Hyaluronate Sterile Sodium (HA) solution indicated for use in the topical management of chronic and acute wounds including pressure ulcers, first & second degree burns, surgical wounds, venous stasis ulcers, and autograft procedures. For use on cats, dogs, and horses (not intended for food animal use).

DIRECTIONS

1. Cleanse wound/surgical site.
2. Withdraw the HA solution.
3. Dispense topically onto the wound site.
4. Cover wound with non-adherent dressing.
5. Change dressing as needed with repeated applications of Transcend™

PART NUMBER

Net Contents: 5 Vials 1003-07-00



www.kineticvet.com