

## ■ VELFAST®


Transparent colourless polypropylene E-collars with double neoprene edging.

- Flexible, silent peripheral neoprene edging, absorbing impacts and noise. This offers more comfort for the animal and faster adaptation to wearing the E-collar.
- Atraumatic neoprene edging arou
- Simple, quick velcro attachment offering excellent tear resistance.
- Velcro strip adjustment according to neck circumference.
- Engraved anti-scratch lines.
- Reinforced with neoprene lining.
- Colour-coded according to size.
- Available in 7 sizes.

Supplied by 12 in cardboard boxes.



Code x 12 VELFAST	Size (A-B)	Neck size (B-C)		O/Pack.
		min.	max.	
306450	7,5 cm	14 cm	17 cm	x 10
306452	10 cm	18 cm	23 cm	x 10
306454	12,5 cm	24 cm	29 cm	x 10
306456	15 cm	30 cm	36 cm	x 10
306458	20 cm	37 cm	43 cm	x 10
306460	25 cm	44 cm	50 cm	x 5
306462	30 cm	51 cm	58 cm	x 5
306468	Set (2x7 sizes, from 7.5 to 30 cm)			x 5

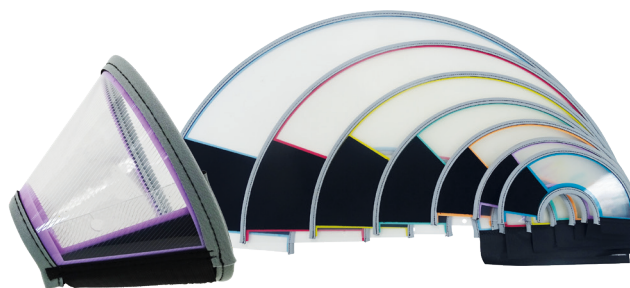


Ultra-quick fastening

COMFORTABLE | LARGE ADJUSTMENT AREA

Features	Benefits	Velfast	Competitors (Kong)
Adjustment	<i>Around the neck of the pet</i>	xxx	x
Speed	<i>Ultra-quick attachment system</i>	xxx	xx
Durability	<i>Double neopren edges</i>	xxx	x
Animal comfort	<i>Atraumatic</i>	xxx	x
Owner comfort	<i>Gentle on legs and furniture</i>	xxx	x
Convenience	<i>Easy storage</i>	xxx	x
Number of sizes		7	4

Reading charts	
xxx	Excellent
xx	Good
x	Basic
-	Poor, N/A

## VELCRO FASTENING SYSTEM

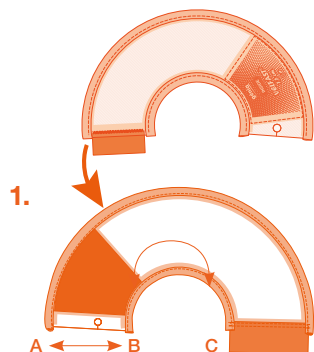

### ASSEMBLY

Position the Velfast® such that the marker indicating 'inside' is located on the animal's skin (1).

Adjust the Velfast® around the animal's neck; you should be easily able to pass your finger between the e-collar and the skin (2).

Superimpose and adhere the velcro strips using all available surface area, without exceeding the limit (3).



HOSPITALIZATION

Contact Génia today for your practice needs and over 2,500 premium SKUs under 9 product ranges: injection, surgery, orthopedics, hospitalization, protection, capture, cleaning, livestock, grooming.

Sales: 917-634-6867 - Customer Service: 646-866-3550